

Getting Started with the µCor™ System

1

Gather Components


Sensor


Patch



Gateway



Charger



Power Cord

2

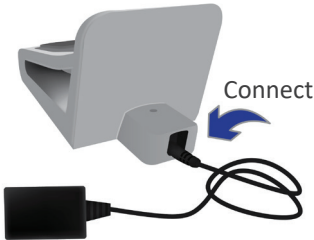Charge the Sensor and Gateway

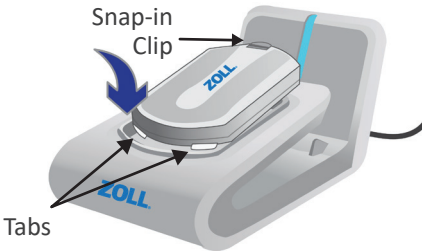
Connect


Snap-in Clip
Tabs




1. Connect Power Cord to Charger.

2. Connect Power Cord to power outlet.


3. Place Sensor into Charger — bottom Tabs should go in first.


4. Place Gateway into Charger.

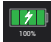System is ready to use when both Sensor and Gateway battery icons display as fully charged  on the Gateway screen.

3

Prepare Your Skin

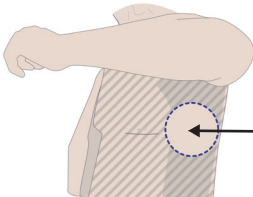
Body Left Side Target Area

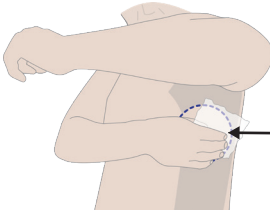
Scrub with Two Preparation Wipes


1. Trim hair (if any) on your left side in the Target Area.


2. Scrub the Target Area with one of the provided Preparation Wipes. Repeat with second Preparation Wipe.


3. Allow skin to dry.

4

Prepare Patch

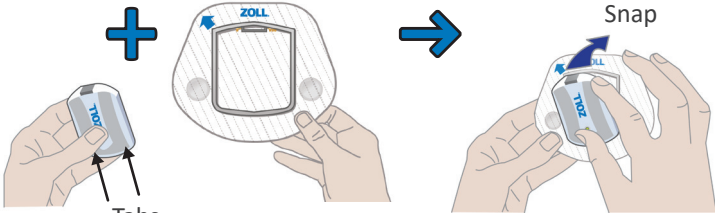
+
Snap
Tabs

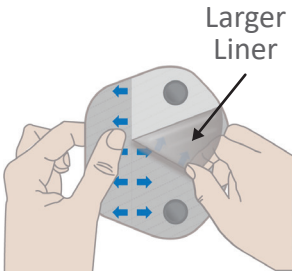
Larger Liner

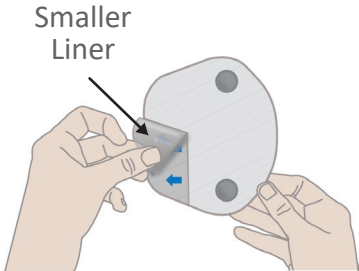

Smaller Liner


1. Install the bottom tabs of Sensor into Patch, then clip Sensor top in place.


2. Peel off the larger liner piece first followed by the smaller liner piece. Try not to touch the exposed adhesive.

5

Place Patch on Body

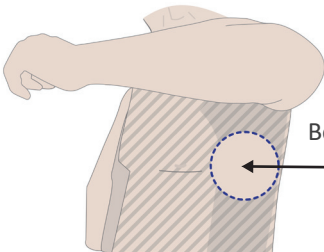
Body Left Side Target Area

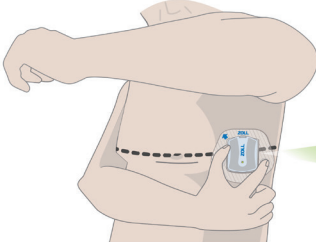

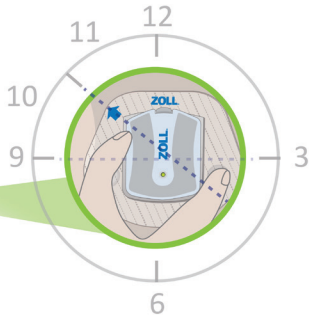



1. Stand in front of a mirror.


2. Raise your left arm to shoulder height.

3. Place Patch about 2-3 fingers wide below your underarm, so that ZOLL is horizontal on the top of Patch.

4. The arrow  should point between 10 and 11 o'clock.

5. Make sure Patch is smooth and flat on your skin.

6. A green light  displays on Sensor within 30 - 60 seconds.

If you see a Green Light  you have successfully set up your system.
The Sensor is now transmitting your data to ZOLL.


If NO Green Light, call ZOLL at 1.888.592.3798

Patient Responsibilities

Every Day


- Wear the Patch with the Sensor at all times, even in the shower and when sleeping.
- Carry the Gateway, keeping it within 30 feet of the Sensor.
- Charge the Gateway overnight while you sleep (at least 3 hours) – See 2 Charge the Sensor and Gateway on page 1.

Replace Patch Every 7 Days

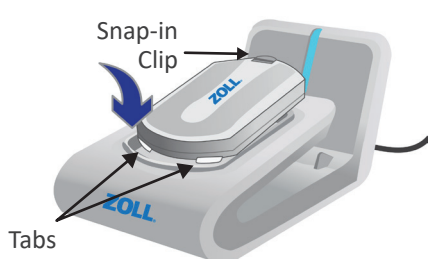


1. Press Sensor snap-in clip to remove Sensor from Patch.


2. Place Sensor in Charger.

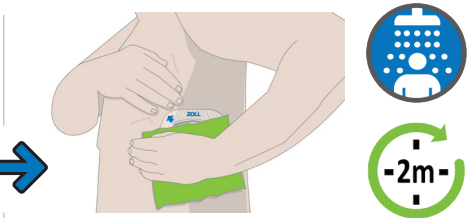

3. Hold a warm wet washcloth against the Patch to get it wet for approximately 2 minutes. This may be done in the shower. To start peeling the Patch off, pull and hold your skin down with one hand.

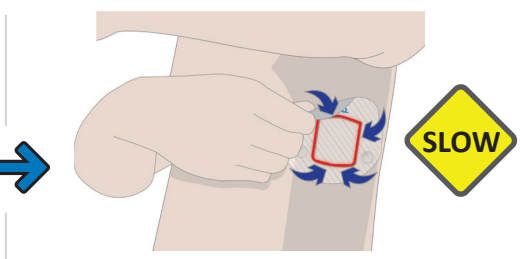

4. Slowly peel each corner of Patch toward the center until only the area under frame (area outlined in red) remains attached to your body.

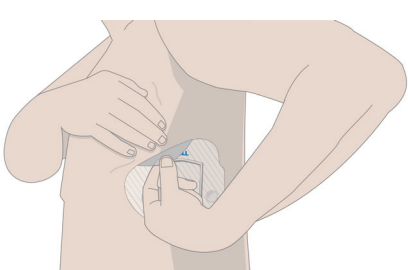


5. Slowly peel the remaining Patch while holding your skin against your body as tightly as possible.


6. Discard Patch. Do not reuse.

7. Once Sensor is fully charged, repeat steps 3 through 5 starting on page 1 using a new Patch. Be sure to apply the new Patch as close to location of the previous Patch as possible.


Record a Symptom & Activity

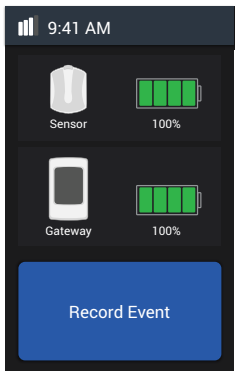

1. Select Record Event.

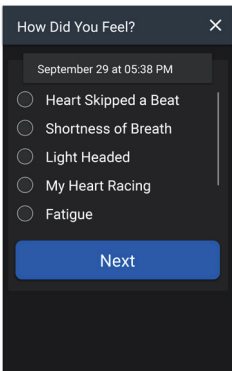

2. Select a symptom that describes how you feel.
3. Select Next.

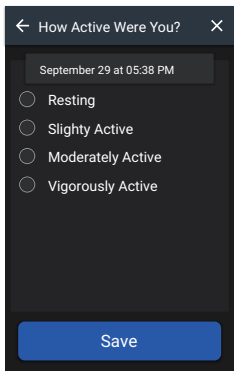

4. Select your activity level.
5. Select Save.



For the Indications for Use statement and a complete list of precautions, refer to the Patient Instruction Manual.



WARNINGS

- Do not use μ Cor™ System if you have allergies or skin sensitivities to electrode hydrogel and/or acrylic based adhesives. Irritation such as redness or itching may develop.
- Do not use μ Cor System if you have skin breakdown in areas where device (Patch + Sensor) placement is required, as this can cause further injury to your skin.
- Do not use the μ Cor System if you have significant swelling on the left side of your chest where the device is applied. Significant swelling may adversely affect device function.

If you have any questions, contact ZOLL
24 hours a day, 7 days a week: Toll free (USA) 1.888.592.3798



121 Gamma Drive • Pittsburgh, PA 15238 USA

Rx Only

Distributed by ZOLL Laboratory Services
Patent: www.zoll.com/patents
© 2024 ZOLL Laboratory Services, LLC. All rights reserved.
ZOLL is a trademark and/or registered trademark of ZOLL Medical Corporation in the United States and other countries.
20B0080 Rev J